
	Bei beschädigter Verpackung nicht verwenden / Do not use if package is damaged / Ne pas utiliser si l'emballage est endommagé. / Non usare se la confezione è danneggiata / No usar si el envase está dañado. / Niet gebruiken als de verpakking beschadigd is. / Må ikke anvendes, hvis pakningen er beskadiget / Ei saa käyttää, jos pakkaus on vaurioitunut / Skäl ikke brukes hvis pakningen er skadet / Använd inte om förpackningen är skadad / Nie używać w przypadku uszkodzonego opakowania / Nepoužívejte, pokud je obal poškozen / Ambalaj hasarlıysa kullanmayın / Μην το χρησιμοποιείτε αν η συσκευασία έχει υποστεί ζημιά لا تستخدم الجهاز إذا كان الغلاف متضرر אין להשתמש אם האריזה ניזוקה
	Beigefügte Anleitung vor Verwendung dieses Geräts bitte sorgfältig durchlesen! / Please read enclosed instructions carefully before you use this device! / Veuillez lire la notice avant d'utiliser ce dispositif !! / Leggere attentamente le istruzioni allegate prima di usare questo dispositivo! / Leer atentamente las instrucciones incluidas antes de usar el dispositivo! / Lees de bijgevoegde instructies aandachtig door voordat u dit hulpmiddel gebruikt! / Les vedlagte anvisninger grundigt, før du anvender dette produkt! / Lue mukana toimitetut käyttöohjeet huolellisesti ennen laitteen käyttöä! / Les vedlagte bruksanvisninger nøye før du bruker denne enheten! / Läs de medföljande anvisningarna noggrant innan du använder pumpen! / Należy dokładnie przeczytać dołączoną instrukcję przed użyciem urządzenia! / Před použitím tohoto prostriedku si pečlivo prečítajte priložené pokyny! / Bu cihazı kullanmadan önce lütfen ilişikte verilen talimatları dikkatle okuyun! / Διαβάστε προσεκτικά τις εσωκλειστές οδηγίες προτού χρησιμοποιήσετε αυτήν τη συσκευή! يرجى قراءة التعليمات المرفقة بعناية قبل استخدام هذا الجهاز! אנא קרא בעיון את ההוראות המצורפות לפני השימוש בציוד זה.
	Nicht bei Sichelzellen-Anämie oder Knochenmarktumoren verwenden. / Do not use in case of sickle cells anaemia or bone marrow tumors. / Ne pas utiliser en cas de drépanocytose ou de tumeurs de la moelle osseuse. / Non usare in caso di anemia falciforme o tumori del midollo osseo. / No usar en caso de anemia drepanocítica o de tumor medular. / Niet gebruiken bij sikkelcelanemie of beenmergtumoren. / Må ikke anvendes i tilfælde af seglcelleanæmi eller knoglemarvstumorer. / Ei saa käyttää sirppisoluanemian tai luuydinkasvainten yhteydessä. / Skäl inte brukas i tillfällen med sigdcellanemi eller benmargstumorer. / Använd inte vid sicklecellanemi eller benmärgstumörer. / Nie stosować w przypadku niedokrwistości sierpowatokrwińkowej lub nowotworów szpiku kostnego. / Nepoužívejte v prípade srpkovité anémie a nádoru kostní dřeně. / Orak hücreli anemi veya kemik iliği tümörleri mevcutsa, kullanmayın. / Μην το χρησιμοποιείτε σε περιπτώση δρεπανοκυτταρικής αναιμίας ή νεοπλασιών του μυελού των οστών. لا تستخدم الجهاز في حالة فقر الدم المنجلي أو أورام نخاع العظام. אין להשתמש במקרים של אנמיה חרמשית או שאתות של מח העצם.
	Gerät trocken lagern / Store device in a dry place / Conserver dans un endroit sec. / Conservare il dispositivo in un luogo asciutto / Guardar el dispositivo en un lugar seco / Bewaar het hulpmiddel op een droge plek / Opbevar produktet på et tørt sted / Säilytä laite kuivassa / Oppbevar enheten på et tørt sted / Förvara pumpen på en torr plats / Urządzenie przechowywać w suchym miejscu / Uchovávejte na suchém místě / Cihazı kuru bir yerde muhafaza edin / Φυλάσσετε τη συσκευή σε ξηρό μέρος احتفظ بالجهاز في مكان جاف. יש לשמור את הציוד במקום יבש.
	Zerbrechlich / Fragile / Fragile / Frágil / Breekbaar / Skrøbelig / Sårkäv / Skjær / Ömtålig / Delikatne / Křehké / Kirilgandir / Εύθραστο تربيع العطب שביר

WORLDWIDE
DISTRIBUTION
NETWORK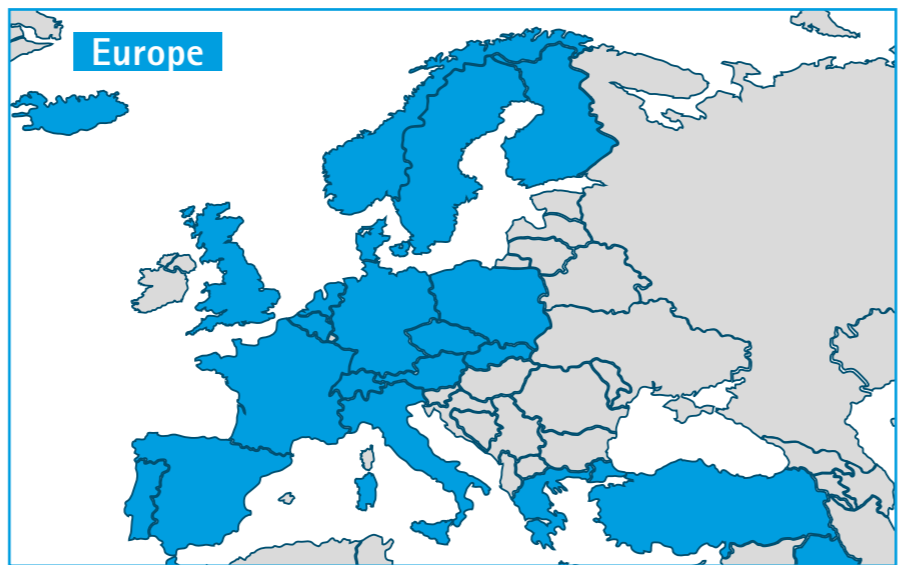

- Deutsch
- English
- Français
- Italiano
- Español
- Nederlands
- Dansk
- Suomi
- Norsk
- Svenska
- Polski
- Čeština
- Türkçe
- ελληνικά

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ACTIVE *Erection System*^{NT®}

Important:

Please read these instructions for use before using the erection aid. Keep the instructions for use! Please watch the animation on the enclosed DVD.

Instructions for Use

When the penis is stiff, this is called an erection. If a man has problems getting an erection, we speak of erectile dysfunction. The **ACTIVE Erection System^{NT®}** is a device that helps you to get your penis stiff. Therefore in this booklet the **ACTIVE Erection System^{NT®}** is also referred to as an erection aid.

What are the causes of erectile dysfunction?

A variety of causes can lead to erectile dysfunction. The most common ones include diabetes, thyroid disease, liver or kidney disease, Treatment of hypertension, cardiovascular diseases. Prostate surgery often also leads to erectile dysfunction. Depression or exhaustion can also lead to erectile dysfunction.

Sometimes erectile dysfunction is a side effect of medication, e.g. drugs against high blood pressure. Smoking or drinking too much alcohol is another possible cause.

The ACTIVE *Erection System*^{NT®} comprises the following parts:

- 1 pump head [1]
- 1 transparent cylinder [2]
- 1 large grey ring loader [3]
- 1 small grey ring loader [4]
- 1 grey cone [5]
- 4 constriction rings in differing sizes [6]
- 1 tube of lubricant [7]
- 3 size AAA batteries [8]
- 1 set of instructions for use with DVD
- 1 bag

How to prepare the application?

All product-related illustrations can be found on the cover flap of this booklet

- Place all parts at hand (Fig. 1)
- Open the pump head as shown in Figure 2.
- Fold up the flap with the nameplate.
- Insert the 3 batteries, paying attention to the marking, as shown in Figure 3. Carefully fold down the flap again, making sure that the nipple on the inside of the flap does not break off.
- Turn the cap back onto the pump head. (Fig. 4)
- Press the pump head onto the larger opening of the cylinder, as shown in Figure 5.
- Plug the large ring loader onto the other side of the cylinder. (Fig. 6).
- Insert the small ring loader into the large ring loader.
- Insert the cone into the small ring loader. (Fig. 7)
- Apply some lubricant to the tip of the cone. (Fig. 8)
- First, place the largest constriction ring (20 mm)

on the top of the cone. Slide the constriction ring over the cone onto the small ring loader. Now take off the cone. (Fig. 9)

– Now the erection aid is ready for use.

How to use the erection aid?

- Remove the pubic hair directly on the penis and around the root of the penis.
- The device must fit the skin tightly.
- Apply some lubricant to the edge of the ring loader. (Fig. 10)
- Introduce the penis through the ring loader into the cylinder. (Fig. 11)
- Press the device tightly to your body.
- Press the (+) "on" button on the pump head for a few seconds. (Fig. 12)
- The motor in the pump head starts and begins to suck air from the cylinder.
- When you press the (+) "on" button again, the pumping will become stronger.
- The device has 4 levels of suction intensity.
- When you press the (-) "off" button on the pump head, the pumping will become weaker. (Fig. 12)
- When you keep the (-) "off" button pressed for a few seconds, the erection aid will switch off. (Fig. 11)

Do you see a good erection in the cylinder?

- Then switch off the erection aid and slide the constriction ring from the ring loader onto the erect penis.
- Slide the constriction ring quickly and as far as possible down to the base of the penis. (Fig. 13)
- The "air" button on the pump head is marked with a dot. (Fig. 14)
- When you press the "air" button, the cylinder is bled (i.e. air is let in, reducing the vacuum).
- Thus air enters the cylinder, and you can easily detach the erection aid from the body.
- The constriction ring on the penis causes the penis to remain rigid. (Fig. 15)
- Now you can have sex.

Important: Never wear the constriction ring on the penis for more than 30 minutes. Do not fall asleep with the constriction ring on the penis!

After sex:

- Briefly pull the constriction ring apart on the flaps.
- Wait until the blood has flown back from the penis into the body.
- Then pull the constriction ring off the penis. When you use the erection aid, you will have no visible ejaculation upon orgasm (climax). (Fig. 16)
- The constriction ring retains the semen (ejaculate) in the penis.
- Once you remove the constriction ring, the semen will flow out of the penis.
- The temporary retention of the semen in the penis is not harmful.
- After using the erection aid, clean it as described in Section "How to clean the erection aid?"
- Dry the parts.
- Place all parts back into the bag.

How does the erection aid work?

The pump head sucks the air around the penis out of the cylinder. This creates a negative pressure (vacuum). This causes the blood vessels in the penis to fill with blood, and the penis stiffens. As the cylinder is transparent, you can see whether the erection is hard enough. In most cases, pumping for 1 to 2 minutes will be sufficient to produce an erection.

Application notes

Application of this erection aid is easy. Nevertheless, it is better to try the erection aid initially alone and unhurried. Then you can show the erection aid to your partner. Once you both know how to use the erection aid correctly, you can enjoy sex again. Even at an advanced age, in illness or physical limitation.

How to select the proper constriction ring?

To keep your erection as long as possible, you need to choose the proper constriction ring. If the constriction ring is too large, your penis will not remain stiff for long. If the constriction ring is too tight, it may cause pain. In case of pain, take off the constriction ring and try a larger constriction ring. If you use the erection aid for the first time, it is best to start with the largest constriction ring. If the constriction ring is too large, try a smaller one, until you find the right size for yourself.

The erection aid includes 2 **ring loaders**.

The smaller one is slightly narrower, the larger a bit wider. If the smaller ring loader is too narrow, just use the larger one. The two ring loaders are suitable **for all constriction rings**. Thus, it does not matter which constriction ring size you are using. If both ring loaders are too narrow, you can slip the constriction ring directly onto the cylinder. The erection aid also includes the **cone**. It helps you to slip the constriction ring more easily onto the ring loaders. Alternatively, you can slip the constriction ring manually onto the ring loaders without using the cone. Use only the supplied constriction rings. From time to time, the constriction rings need to be replaced. They can be ordered from **service@medintim.de**

When the motor of the erection aid is switched on, 2 LEDs are lit. If the LEDs are flickering, the batteries are low and need to be replaced. Use only 1.5-volt batteries of size AAA Alkaline. Do not use rechargeable batteries. Always use 3 identical and new batteries.

Never use this erection aid if you have drunk alcohol or have taken recreational drugs.

Do not continue using the erection aid if it does not work properly.

Do not use the erection aid either if it is visibly damaged. In both cases, please send the erection

aid directly to the manufacturer, **KESSEL medintim GmbH**.

During the warranty period, repairs are free. If the device is not working or damaged because you have operated it incorrectly, you will always have to pay for the repairs.

If you do not live in Germany, please contact the dealer from whom you bought the erection aid. Use only lubricants that can easily be washed off with water.

Do you have any questions?

Feel free to call this number:

+49 (0) 6105 20 37 20

Or send an email to:
service@medintim.de

What side effects can the erection aid have?

Do you suffer from cardiovascular disease?

Then talk to your doctor or health care professional before using the erection aid for the first time.

The following side effects are possible:

You can get a bruise (haematoma) at the base or at the top of the penis.

Sometimes there may even be slight tissue injury, when the penis was not erect for a long time.

However, that is not dangerous.

When may the erection aid not be used?

Do not use the erection aid if you have any of the following conditions:

- Diseases that affect the red blood cells, e.g. sickle cell anaemia
- Blood cancer (leukaemia)
- Bone marrow tumours
- Diseases that affect blood clotting (coagulation)

- Diseases that cause bleeding or prolonged erections
 - Other diseases in which sex is not recommendable
- If you are not sure whether you can apply the erection aid without any problem for your health, ask your doctor or health care professional.

How to clean the erection aid?

Before cleaning, take the pump head off the erection aid.

The pump head may be cleaned only with a moist cloth from the outside.

All other parts can be easily cleaned with warm water and mild soap.

Do not use aggressive detergents.

How to recycle the erection aid correctly?

Dispose of the batteries into the special waste for batteries. Dispose of the pump head into the special waste for small electrical appliances.

All other parts can be disposed of into the household waste.

Warranty

Almost all parts are warranted for 2 years, as prescribed by applicable law.

Wear parts, such as constriction rings, are not covered by warranty. They must be replaced from time to time.

The warranty does not apply if you damage the erection aid or use it incorrectly.

The pump head is sealed inside and may be opened only by the manufacturer or an authorised service partner. You should send in the erection aid, if possible, every 3 years for technical inspection.

What material are the parts made of?

- Cone: Plastic (acrylonitrile butadiene styrene, ABS)
- Small and large ring loader: Plastic (acrylonitrile butadiene styrene, ABS)
- Cylinder: Plastic (polycarbonate)
- Constriction rings: Biocompatible Thermoplastic Elastomer (TPE)
- Bag: Nylon®
- Batteries: 3x Alkaline AAA, 1.5 volts

During application, there is no danger from electromagnetic radiation.

You can reorder all the parts and the lubricant.

To get the current reordering list, please contact **service@medintim.de** or **info@savantini.com** for UK (+44 0142 888 785) or your dealer.

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FAQ AES

Medical questions on use of the system

Can any man use the ACTIVE Erektion System^{NT}?

Yes, usually. It will help any man to achieve an erection again and maintain it, whether he is young or old, healthy or ill.

Can I use the ACTIVE Erektion System^{NT} if I have diabetes?

Yes, it is especially useful for diabetic patients, as other methods are often not very helpful.

Can I use the ACTIVE Erektion System^{NT} if I have high blood pressure?

Yes, the vacuum pump is very effective in helping men who are taking medications for high blood pressure to achieve an erection. In fact the cost of the unit is usually reimbursed by the statutory health insurance agency.

Can I use the ACTIVE Erektion System^{NT} if I am taking anticoagulants (blood thinners) such as warfarin?

Generally, yes. Using a vacuum pump is not a health risk for users of anticoagulants (blood-thinning medications). However, there could be (strong) bruising, which might make continued use of a vacuum erection appliance unwise. Serious or permanent injury is not possible.

Can I use the ACTIVE Erektion System^{NT} after prostate surgery?

Generally, yes. However, we usually recommend the MANUAL Erektion System in this case. This pump produces a vacuum (suction) rather more quickly and effectively. We would be happy to advise you. The cost of this unit is usually reimbursed by the statutory health insurance agency in cases such as yours. Your doctor can give you a prescription for this as a therapeutic appliance.

Can I use the ACTIVE Erektion System^{NT} after radiotherapy (prostate)?

Yes, use of a vacuum pump such as the ACTIVE Erektion System^{NT} is also useful and effective after radiotherapy of the prostate.

The cost of this unit is usually reimbursed by the statutory health insurance agency in cases such as yours. Your doctor can give you a prescription for this as a therapeutic appliance.

Can I use the ACTIVE Erektion System^{NT} after a radical prostatectomy?

Generally, yes. However, we usually recommend the MANUAL Erektion System[®] in this case. This pump produces a vacuum (suction) rather more quickly and effectively.

The cost of this unit is usually reimbursed by the statutory health insurance agency in cases such as yours. Your doctor can give you a prescription for this as a therapeutic appliance.

Since my prostate operation, I sometimes find it difficult to hold my urine in or have slight involuntary leakage. Can I still use a vacuum pump?

Yes, in this case we recommend the MANUAL Erektion System[®], because this device is easier to clean if any urine gets into the cylinder while pumping.

Is it there an increased risk of thrombosis (blood clots) if I use the vacuum pump?

No, treatment with the vacuum pump does not affect your risk of thrombosis (blood clots).

I am over 70 – can I still use the ACTIVE Erektion System^{NT}?

Yes, of course. We have many customers who are over 70 and even in their mid 80's. In fact our oldest customer, who we spoke to personally, was 92. Sexual activity has no age limits. Vacuum pumps are a simple and practical aid.

Can I use the ACTIVE Erection System^{NT} if I have a curved penis (Peyronie's disease)?

Yes, as long as the curvature of the penis does not prevent you from inserting it into the cylinder.

We have received several positive comments from patients, who said that regular use of the vacuum pump reduced the curvature of the penis.

Can I use the ACTIVE Erection System^{NT} if I have mild Parkinson's disease?

Yes, you can use a vacuum pump if you have Parkinson's disease.

Can I use a vacuum pump if I have MS (multiple sclerosis)?

This neurological disorder often results in weak or inadequate erections, lack of orgasm or failure to ejaculate. Use of the ACTIVE Erection System^{NT} can help to solve these problems. A vacuum pump can be used.

With which diseases can the ACTIVE Erection System^{NT} (vacuum pump) not be used?

In sickle cell anaemia, leukaemia, bone marrow cancer or other diseases that alter blood clotting; this could result in bleeding and prolonged erections. If there are any other medical reasons not to engage in sexual activity, the appliance should not be used.

I have a bruise on my penis after using the vacuum pump. Is this dangerous?

No, that sometimes happens - particularly with diabetes. The bruise will disappear in a few days.

Can my penis get bigger when I use the vacuum pump?

Yes, this is possible, because the vacuum increases blood flow to the penis and expands the erectile tissue.

How often can I use the ACTIVE Erection System^{NT}? Is it possible to use it several times a day?

You can use the unit as often as you like. There is no risk of injury.

Can I generate too much suction or injure myself by using the vacuum pump?

No, the strength of the vacuum cannot exceed a set value and you cannot injure yourself.

Sometimes there is a feeling of tension when I use the unit - I find this unpleasant.

Select a lower level of suction on the unit (press the minus button) and perhaps let a little air in (inlet button in the middle). With the vacuum pump, the quality of an erection is better if it is developed slowly.

If I use the ACTIVE Erection System^{NT}, will I have normal sex with an orgasm?

Yes, with the artificial erection you can have normal sex or masturbate for up to half an hour (30 min.). You can still have an orgasm as before.

Will I still ejaculate if I use the vacuum pump?

You can still ejaculate, but the semen is held in by the constriction ring. After taking the constriction ring off, the ejaculate will flow out. A visible „spurt“ at orgasm is not possible.

Isn't it a problem if the semen can't flow out at orgasm?

No, the vessels can store it and release it later.

Can I still father a child if I use the ACTIVE Erection System^{NT}?

If you want to have a child, you can use a special ring, the ACTIVE LOOP RING (ALR). This ring can be opened after orgasm with a small tab, so that the semen can flow out while the penis is still in the vagina.

If you do not want to father a child, the constriction rings are not a safe means of contraception, and a safe form of contraception should be used.

I use a method in which I inject a drug into the shaft of the penis. Can I still use the ACTIVE Erection System^{NT}?

Yes, there are quite a number of users of this medicinal method for treating erectile dysfunction. They start by developing an erection with the ACTIVE Erection System^{NT}, and then inject the drug into the erect penis, which is now well supplied with blood. In this way the drug is distributed better through the erectile tissue.

My partner thinks that my penis feels colder since we started using the pump.

Yes, this is possible, because the constriction ring reduces blood flow.

Solution: You can apply a warm gel beforehand, or briefly wrap hot towels or a hot water bottle around the penis.

Practical questions on use of the system

I can't put the parts of the ACTIVE Erection System^{NT} together.

First have another look at the steps in the instructions. Have you watched the DVD that came with the unit? If it doesn't work, we can help you further on the phone at +49 6105 203720.

The unit doesn't work. What should I do?

First of all, check whether all three batteries are properly installed.

Check to see whether all the batteries are fully charged.

What should I do if the motor is running, but it's not producing a vacuum (doesn't suck).

You can test to see if the motor is generating enough suction. Hold the motor with its cylinder on a flat part of your body, e.g. your thigh or your abdomen. If the cylinder sucks onto your skin and makes a „plop“ sound when you take it off, the unit is working.

Try it again on the penis and make sure that there is a tight seal between the cylinder and the shaft of the penis. A little lubricant (e.g. Sylk or KY Jelly) helps to make a good seal between the cylinder and the shaft.

What can I do if the motor is running, but there is no vacuum?

Send us the motor in a prepaid package and we will check it. You have a two year warranty. Don't forget to give the address of the sender!

The motor stops and starts and the lights flicker on and off. What's going on?

The batteries are dead. Please replace the batteries.

How can I remove the motor from the cylinder? They won't come apart.

Try again using a rotating movement. If that doesn't help, you can hold the cylinder under hot water and then try again to release the motor.

If this still doesn't help, you can send us the motor with the cylinder.

My erection doesn't last long, even with the constriction ring.

Try using a smaller, tighter constriction ring.

My erection doesn't last long, even with the smaller constriction ring.

It is possible to use two constriction rings together. There are also specially made elongated guide sleeves, which you can find on list of spare parts. We would be happy to advise you.

The erection is still good as long as I'm pumping with the cylinder, but as soon as I put the constriction ring on the erection gets weaker. Then I can't really have sex.

There is a little trick that will help you to maintain a really firm erection by „re-pumping“. After slipping the constriction ring on, you use the pump again. Remove the grey guide sleeves from the cylinder. Now put just the pump with the cylinder over your penis, which already has the constriction ring in place. Press the cylinder firmly against the shaft and produce a vacuum around the penis again. Blood can now flow into the penis, but it can't flow out again. After you take the cylinder off, the erection will now be really firm and at right angles to your body.

I use the large guide sleeve, but it is difficult to pull the glans out of the cylinder through the large guide sleeve when the penis is erect.

Don't use the guide sleeve at all, just stretch the constriction ring directly onto the cylinder. Then it should work.

When my penis is erect, the glans is up against the motor. Is there a longer cylinder?

Yes, there is cylinder which is about 5 cm longer, which you can find in our parts list. We would be happy to advise you.

My penis completely fills the cylinder when it's erect, and it's difficult to pull it out. Is there a larger cylinder?

Yes, there is our „Special Plus“ system. This is a larger and longer cylinder which fits onto the motor with an adapter. We would be happy to advise you.

The suction produced by my ACTIVE Erection System^{NT} is getting weaker and weaker, is there anything I can do?

Yes, the white membrane on the motor may be clogged up. This sometimes happens with lubricant residues. Etc. You can clean the white membrane on the motor carefully: take a cotton bud and moisten it. You can then use it to clean the outside of the Goretex[®] membrane carefully. Note: don't apply too much pressure, or you could make a hole in the membrane. If this is doesn't work, you can send the motor to us for repair.

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Guidelines and manufacturer's declaration – electromagnetic immunity			
The Active Erection System is intended for use in the following electromagnetic environment. The customer or user of the Active Erection System should ensure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidelines
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Compliant	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz 80% AM (1 kHz)	Compliant	The field strength of stationary RF transmitters outside of the shielded location, determined by an electromagnetic site survey, should be under 3 V/m.
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not applicable (no electrical cables)	
Electrical fast transients IEC 61000-4-4	± 2 kV supply output ± 1 kV input/output power	Not applicable	The medical electrical device is operated exclusively by battery power.
Voltage output IEC 61000-4-5	± 1kV Differential mode ± 2kV Common mode	Not applicable	
Mains frequency magnetic field IEC 61000-4-8	50 Hz 3 A/m (root mean square)	Compliant	Magnetic fields should be within the normal range of a typical location.
Voltage dips, short interruptions and voltage variations of the power supply input lines IEC 61000-4-11	> 95% reduction 0.5 cycles	Not applicable	The medical electrical device is operated exclusively by battery power.
	60% and 30% reduction 5 and 25 cycles		
	> 95% reduction 250 cycles		

Guidance and manufacturer's declaration – electromagnetic emissions		
The device Active Erection System is intended for use in the following electromagnetic environment. The customer or user of the device Active Erection System should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The medical electrical device uses RF energy only for internal functions. For this reason, the RF emissions are very low and it is not likely that they would cause interference to electronic equipment nearby.
RF emissions CISPR11	Class B	Compliant
Harmonic currents IEC 61000-3-2	Not applicable	The medical electrical device is operated exclusively by batteries.
Voltage fluctuations/ flicker emissions	Not applicable	

This device corresponds to the provisions of EC directive 93/42/EEC.

The Active Erection System meets the standards regarding electromagnetic emissions. According to current knowledge, the device is safe for use provided it is handled properly and according to the directions in these instructions for use.

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Bu ürün 93/42/AET sayılı AB Yönergesi'ne uygundur.